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EN | INSTRUCTIONS FOR USE

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IMPLANT INSTRUCTIONS  
FOR USE

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INSTRUCTIONS FOR USE

IMPORTANT INFORMATION

Read these instructions for use carefully before using the product.

This document contains basic information for the use of the DentPross® original dental implant systems, from now on referred to as "DentPross® dental implants" or simply "DentPross® products". This document has been written as a quick consultation guide for the practitioner responsible for the treatment, from now on referred to as "User", and does not provide sufficient technical specifications for the correct use of the DentPross® products. It is not an alternative or substitute for specialised training and professional clinical experience.

The DentPross® products must be used carrying out sufficient planning of the treatment and rigorously following the surgical and/or prosthetic protocols set by the manufacturer. Read the specific surgical and prosthetic protocols for each product carefully, as well as the instructions for use and maintenance, before using any DentPross® product. You can consult them on our website [www.dentpross.es](http://www.dentpross.es) or request them from your official authorised distributor.

## 1. Description of the system

The DentPross® products are composed of dental implants, abutments or prosthetic components and surgical and prosthetic instruments.

NetPlus Dental Abutments SLU has created a wide range of dental implant systems for the carrying out of dentistry treatments in accordance with the existing scientific bibliography and current clinical standards. NetPlus Dental Abutments SLU develops and commercialises latest-generation implant products and a wide range of restorative solutions to respond to the particular needs of each patient.

## 2. Description of the products

### Dental implants

The DentPross® dental implants are manufactured in grade 4 biocompatible titanium in accordance with regulations ASTM F67 and ISO 5832-2, with a surface treatment. The DentPross® dental implants are manufactured with different diameters, lengths, shapes and connections depending on their intended purpose.

### 3. Indications for use

It is essential to previously evaluate the patient by means of a clinical and imaging diagnosis and study of models, as well as to determine the possible contraindications for the treatment, prior to carrying out a treatment with DentPross® products. A correct treatment with dental implants must be based on a diagnosis, a treatment plan and a design of the prosthetic rehabilitation which meets the needs and expectations of the patient.

### Implants

The DentPross® dental implants are indicated for rehabilitation both in upper and lower jaw, and are aimed at recovering the function of chewing, as well as providing improved occlusion and distribution of occlusal load, phonation, aesthetic appearance, prosthetic stability and maintenance of the alveolar process, as well as a positive psychological effect. They are supplied in different designs and with different lengths and diameters to adapt to the different clinical cases.

The DentPross® dental implants are single-use products.

### Important

Carefully consult the clinical indications for each model of implant in the corresponding catalogues and surgical protocols, before planning a surgical intervention. Do not use the DentPross® dental implants in dental positions or situations which are not described as indications in their catalogues or surgical protocols.

### 4. Contraindications

It is necessary to carry out a preoperative medical examination of the patient and to determine if absolute and/or relative contraindications or risk factors for the treatment with dental implants exist. The user is responsible for evaluating the potential risks and benefits of the implant treatment and for taking the pertinent decisions depending on the case. The DentPross® products must not be used on patients who lack the minimum medical conditions for receiving a prosthetic treatment and rehabilitation with dental implants.

### Relative contraindications

Age of the patient - Uncontrolled systemic disease - Poor oral hygiene - Active local infection - Pregnancy - Therapy with bisphosphonates - Sjogren syndrome - Uncontrolled endocrine disease - Radiation to head and neck - Allergy to stainless steel - Stress - Psychological factors - Valve prostheses - Uncontrolled periodontal disease - Smoking - Alcoholism - Drug addiction - Addictions in general.

### Absolute contraindications

The manufacturer does NOT recommend the implant treatment in the following cases: Cardiovascular diseases or disorders – Haemophilia – Cirrhosis of the liver – Terminal pathologies – Epilepsy – Neoplasms under care with chemotherapy – Blood dyscrasias – Cancer – Uncontrolled diabetes – Untreated periodontitis – Infectious diseases – Leukaemia – Alterations of the immune system – Treatment with immunosuppressives – Allergy to titanium – Active treatments with radiotherapy – Indefinite treatments with corticosteroids – Treatments with anti-coagulants – Involuntary muscle disorders – Serious liver disorders – Endocrine disorders – Psychiatric and/or psychological disorders.

The manufacturer will update these contraindications when necessary, based on the information obtained through clinical studies and post-sales information.

### 5. Storage

The DentPross® products must be stored in their original packaging at a temperature of between -10 and 55°C, in a clean, dry place, protected from sunlight and adverse conditions.

### 6. Packaging and supply state

DentPross® dental implants are supplied in a cardboard box with a product identification label wrapped in shrink-wrap plastic film that acts as a security seal against tampering.

This package contains:

- QR code to access these product Instructions for Use. They indicate the reference, batch and manufacture of the implant.

- Thermosealed plastic blister with label on the back which indicates the manufacturer, the full product reference and the batch number. Inside is the implant carrier container (where the implant is housed with its cover screw. In

the case of implants with Mount, it will also contain the carrier and the clinic screw).

The removable labels are used to control the traceability of the implant, sticking one label on the patient's Implant Card and the other for the clinic's record.

The manufacturer guarantees that all its products follow an exhaustive process of manufacture, control and cleaning before being packaged. The correct integrity of the packaging must be controlled before using the DentPross® products, checking that it is not damaged. If there is any defect or damage, it must not be used and must be notified to its official authorised distributor.

### 7. Cleaning and sterilisation

The DentPross® dental implants are supplied sterilised by irradiation with Beta Rays at 25 kGy. If the sterile package is altered, tampered, damaged or broken, the sterility is not guaranteed and the product must not be used. Do not use the product after the expiry date indicated on its package.

The re-sterilisation of the DentPross® dental implants is NOT authorised.

### Important

The plastic blister containing the implant container is NOT sterile, only the inside of the container is sterile; do not place the container in the sterile surgical field.

### 8. Precautions

#### Surgery and Instruments

The surgical protocols describe in detail the precautions and important factors to take into account during the surgical process of fitting the dental implants. Prior to the intervention, it is essential to have consulted the surgical protocol of the product, as well as all the indications about its correct usage indicated in the catalogues of the DentPross® products.

Particular attention must be paid and the necessary measures taken to prevent the instruments, of such small size, from being swallowed or breathed into an airway. All the DentPross® instruments for manual use incorporate retaining elements between each other or for use with dental floss, to avoid accidental ingestion.

It is important to reduce the time of surgery and the damage done to the area receiving the implants, paying special attention to bone trauma and the asepia of the surgical field. The correct preparation of the surgical site is a fundamental factor in the success of the treatment. This process requires that the surgical protocol for drilling set by the manufacturer for each type of implant and the correct use of the surgical drills, both in terms of the number of uses and of the working speeds, be followed rigorously. The surgical drills and taps must be used with constant abundant irrigation to prevent overheating of the bone.

Take special care not to apply excessive forces or torques during the insertion of the implant. It is important to control the insertion torque of the implant, both in insertion with contra-angle, and with insertion with ratchet wrench.

### Attention

When surgical instruments with a limited number of uses are used, this number must not be exceeded, as this would seriously endanger the osseointegration process and therefore the treatment.

The maximum torque for insertion of the dental implants is 50 Ncm. Exceeding the maximum insertion torque indicated for the implants can cause serious damage to the dental implant, its connection, the mount and the clinical screw included. Refer to the surgical protocol for specific mount removal considerations, according to implant connection type and bone type. Refer to the surgical protocol for specific mount removal considerations, according to implant connection type and bone type.

### 9. Warnings

The planning of the treatment and the fitting of the DentPross® dental implants requires specific dentistry training. The specifications of the product alone do not guarantee a proper use of it. The users are recommended to attend specialised theoretical/practical training courses to become familiar with the work techniques and protocols suitable for each product, including biomechanical, radiographical and prosthetic requirements associated with the implant treatment. It is the user's obligation to know and be informed of the current state of the technique for each product and its possible applications.

Before using DentPross® products, it is necessary to know and be familiar with the corresponding surgical and prosthetic procedures. The patient must meet sufficient anatomical and psychological conditions to undergo the treatment with dental implants. The user must ensure the correct planning of the treatment, guaranteeing its safety with a minimum margin for error and respecting the vital oral structures and the general health of the patient.

The procedure for use of the surgical drills, taps and other surgical instruments needed for the fitting of the implants is specified in the surgical protocols corresponding to each type of implant. The fitting of the implant and the prosthetic planning must adapt to the patient's individual conditions, and in particular to a correct distribution of forces. The passive adjustment in the prosthetic rehabilitation structure and the adjustment of the occlusion must be achieved, and the appearance of excessive lateral forces avoided. A treatment with an insufficient number of implants, or an unsuitable choice of size and position to withstand and transmit the loads expected, can cause mechanical failure of the implant, of the abutment or prosthetic components or of the

clinical screws due to overburdening or fatigue and substantial bone loss around the implant.

A qualitatively or quantitatively insufficient alveolar bone, the appearance of infections or diseases in general, and changes in the habits of the patient, are some potential causes of failure of the osseointegration and of the treatment. The lack of bone tissue or soft tissue can give rise to a deficient aesthetic result. An unsuitable prosthetic rehabilitation can cause the failure of the whole rehabilitation treatment.

The reuse of single-use products carries the danger of possible deterioration of their properties. The geometry of the product may have suffered damage during previous use or due to an unsuitable use. This implies the risk of surgical or prosthetic failure and/or harm to the patient's health.

Electrosurgery is not indicated due to the conductivity of the dental implants.

#### 10. Adverse or side effects

Many adverse or undesired effects can present themselves in current implantology. These are documented in the specialised scientific bibliography and published in journals and books specialised in the dentistry sector.

However, the most relevant are:

Post-operative discomfort – Localised pain – Localised inflammation – Haematomas – Haemorrhages – Localised or systemic infections – Difficulty speaking – Bone loss and/ or fracture – Loss of the implant – Damage to adjacent teeth – Fractures to the implants and/or prosthetic components – Fracture of instruments – Injury to adjacent anatomical structures (inferior alveolar nerve or maxillary sinus).

The manufacturer will update these side effects when necessary, based on the information obtained through clinical studies and post-sales information.

#### 11. Information on responsibility, safety and guarantee

The indications for use and handling of the DentPross® products are based on the international published bibliography, the current clinical standards and our clinical experience with our products, and therefore should be understood as indicative general information. The handling and use of the DentPross® products, being out of the control of the manufacturer are sole responsibility of the user of them. The manufacturer, its subsidiaries and/or official authorised distributors assume no responsibility, whether express or implicit, full or partial, for the possible harm or damages which may result from the improper handling of the product or from any other event not specified in its protocols and manuals for the correct use of its products.

The user of the product must ensure that the DentPross® product used is suitable for the intended procedure and purpose. Neither these instructions for use, nor the work and product handling protocols, absolve the user of this responsibility. The use, handling and clinical application of the DentPross® products must be carried out by qualified professional staff, with the necessary qualifications in compliance with the legislation in force in each country. The use, handling and/ or application of the DentPross® products, whether fully or in part, at any stage, by unqualified staff or staff without the necessary qualifications, automatically renders any kind of guarantee void and can cause serious harm to the health of the patient.

The DentPross® products form part of their own system, with their own design characteristics and work procedures. This includes the dental implants, abutments or prosthetic components and the surgical or prosthetic instruments. The use of DentPross® products in combination with elements or components from other manufacturers can cause failure of the treatment, cause tissue damage, cause damage to the bone structures, inadequate aesthetic results and serious harm to the patient's health. For this reason, only original DentPross® products must be used.

The clinical professional responsible for the treatment is the sole person responsible for ensuring the use of original DentPross® products and for using them in conformity with the corresponding instructions for use and handling protocols for the whole duration of the implant treatment process. The use of components, instruments or any other non-original DentPross® product used alone or in combination with any of the original DentPross® products will automatically void any kind of guarantee of the original DentPross® products. Consult the conditions of guarantee with your Distributor.

#### Warning

Not all the DentPross® products are available in all countries. Consult their availability.

The DentPross® brand is a registered trademark, as are some of the products mentioned or not mentioned in this document. Biostar Laboratorios SLU reserves the right to modify, change, remove or develop any of the products, prices or technical specifications referenced in this document or in any of its catalogues without prior notice. All rights reserved. The whole or partial reproduction or publication of this documentation, by any means and in any format, is forbidden without the corresponding authorisation in writing from the manufacturer.

#### 12. Magnetic resonance

The safety and compatibility of the DentPross® products in a magnetic resonance environment has not been evaluated, nor have they undergone testing to see if they heat up or migrate in said environment.

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